Upcoming entrained bubbles get eliminated through a holding time of approx. 1 hour with FotoDent® setup.

To avoid damage of material quality do not expose the liquid material to light under any circumstances. Variations from the described manufacturing process may lead to modified mechanical characteristics and colour variations of the FotoDent® setup material.

Possible dangers

Please note the safety instructions for processing FotoDent® setup.

Hazard statements: H318 Causes serious eye damage. H317 May cause an allergic skin reaction. H412 Harmful to aquatic life with long lasting effects.

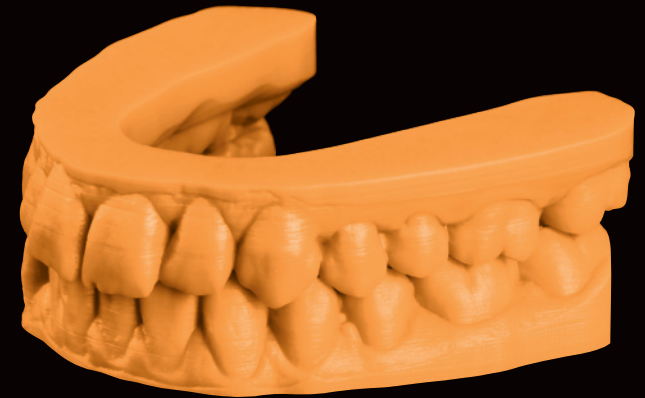
Precautionary statements: P261 Avoid breathing dust/fume/gas/mist/vapours/spray. P280 Wear protective gloves/eye protection/face protection. P273 Avoid release to the environment. P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P310 Immediately call a POISON CENTER/doctor. P501 Dispose of contents/container in accordance with local/regional/national/international regulations.

Contains: 2-Propenoic acid, reaction product with Pentaerythrite, 1,4 Butanediol dimethacrylate, phenyl bis(2,4,6-trimethylbenzoyl)-phosphinoxide

Lot number / Expiry date

The lot number and the expiry date are indicated on each FotoDent® setup packaging. In case of claims please always indicate the lot number of the product. Do not use the product after the expiry date.

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Product description

FotoDent® setup is a light-curing resin for the production of dental models using 385 nm/405 nm LED based stereolithography. For the post curing of FotoDent® setup built parts, the light curing units PCU LED, PCU Evo and FotoDent® Flash are recommended, which are optimal balanced to the production process.

Processing

- During the application of FotoDent® setup we recommend wearing personal protective equipment e. g. suitable gloves, safety goggles etc.
- FotoDent® setup should be well shaken approx. 1 hour before use.
- Carefully fill pour FotoDent® setup into the pre-determined container of the production unit.
- Remove all bubbles with a cleaned object.
- Make sure that FotoDent® setup is tempered in your production unit up to approx. 23 °C.
- Select the product specific parameter settings for your process.
- When the building process is finished a direct post treatment is recommended.
- After the platform is taken up a dripping off time of approx. 10 minutes is recommended.
- Resin coated parts get clean with isopropanol (97 %) within a time of about 10 minutes. For a particular effective cleaning with isopropanol the use of explosion-protected ultrasonic units is recommended.
- Post curing:

PCU LED	8 minutes, 80 % light power
PCU Evo	8 minutes + 1 minute gas pre-flow
FotoDent® Flash	8 minutes + 1 minute gas pre-flow
- Impurity due to the construction or a break of the material and impurity due to operation mistakes cannot be excluded. Thanks to the low viscosity however it is possible to filtrate FotoDent® setup. It is recommended to regularly take out the container of the production unit, to homogenize and filtrate the content.