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#### Hazards identification

- Please observe the safety data sheet when using Sonopal hart Lack Al. Available on request.

#### Intended use

Light-curing lacquer for BTE and ITE hearing systems.

#### Indication

Hearing loss or impaired hearing.

#### Indication for use

Sonopal hart Lack Al is a light curing lacquer for coating BTE earmolds and ITE shells.

#### Contraindication

If an acrylic allergy exists, an allergic reaction may occur.

#### Patient target group

People with impaired hearing ability to wear a hearing aid.

#### Intended users

Qualified personnel from earmold laboratories and hearing aid acousticians.

#### Requirements

Polylux post curing units (Polylux LED, Polylux 100, Polylux 500, Polylux 1000N), PCU LED N<sub>2</sub> und Sonolux Flash Evo.

#### Basic Material Properties

See technical datasheet (TDS). Available on request.

#### Specific Manufacturing Considerations

#### Caution

During the application of Sonopal hart Lack Al we recommend wearing personal protective equipment i.e. suitable gloves, safety goggles etc.

Sonopal hart Lack Al is suitable for lacquering hard BTE earmolds and ITE shells. The lacquer can be applied using a dipping method or with a brush.

#### Processing

#### Preparation

The surfaces of the BTE earmold or the ITE shell should be free from dust and grease residues. Roughening of the plastic surface is recommended for improved wetting behaviour.

#### Further processing

Dip the BTE to be lacquered into the Sonopal hart Lack Al and allow to drip dry. Take care to ensure that no air bubbles form. For ITE shells with integrated electronics, the lacquer should be applied in a very thin coating with a brush.

#### Post Curing

PCU LED N <sub>2</sub>	5 min. 80 <span> </span> % light output (without nitrogen, without vacuum)
Sonolux Flash Evo	3 min. (without nitrogen)
Polylux 1000N	5 min. (without nitrogen)
Polylux LED	5 min.
Polylux 500	5 min.
Polylux 100	5 min.

#### Disposal

Disposal of contents / container in accordance with local / regional / national/international regulations and according to the safety data sheet.

#### Lot number / Expiry date

The lot number and the expiry date are indicated on each Sonopal hart Lack Al packaging. In case of claims please always indicate the lot number of the product. Do not use the product after the expiry date.

#### Cleaning

The lacquered BTE earmolds and ITE shells should be cleaned before first use in patients. We recommend the OtoVita<sup>®</sup> care products series for this. Please observe the separate OtoVita<sup>®</sup> instructions. The use of aggressive cleaning agents can damage the material and the surface.

#### Disinfection

We recommend the OtoVita<sup>®</sup> care products series for disinfection of the lacquered BTE earmold or ITE shell. Please observe the separate OtoVita<sup>®</sup> instructions.

#### Sterilization

Sterilization of lacquered BTE earmolds or ITE shells is not required. Sonopal hart Lack Al cannot be sterilized.

#### Serious incidents

All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is settled.

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