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Intended use

Light-curing lacquer for BTE and ITE hearing systems.

Indication

Hearing loss or impaired hearing.

Indication for use

Sonopal soft Lack E is a light curing lacquer for coating BTE earmolds and ITE shells.

Contraindication

If an acrylic allergy exists, an allergic reaction may occur.

Patient target group

People with impaired hearing ability to wear a hearing aid.

Intended users

Qualified personnel from earmold laboratories and hearing aid acousticians.

Requirements

Post curing unit: Sonolux Flash Evo

Basic Material Properties

See technical datasheet (TDS). Available on request.

Specific Manufacturing Considerations

Caution

During the application of Sonopal soft Lack E we recommend wearing personal protective equipment i.e. suitable gloves, safety goggles etc.

Sonopal soft Lack E is suitable for lacquering BTE earmolds and ITE shells. The lacquer can be applied using a dipping method or with a brush.

Processing

Preparation

The surfaces of the BTE earmold and ITE shell should be free from dust and grease residues. Roughening of the plastic surface is recommended for improved wetting behaviour.

Further processing

Dip the BTE earmold to be lacquered into the Sonopal soft Lack E and allow to drip dry. Take care to ensure that no air bubbles form. For ITE shells with integrated electronics, the lacquer should be applied in a very thin coating with a brush.

Post Curing

Sonolux Flash Evo	3 min. + 1 min. gas pre-flow (with nitrogen)
-------------------	---

Disposal

Disposal of contents / container in accordance with local / regional / national / international regulations and according to the safety data sheet.

Contraindication

If an acrylic allergy exists, an allergic reaction may occur.

Patient target group

People with impaired hearing ability to wear a hearing aid.

Intended users

Qualified personnel from earmold laboratories and hearing aid acousticians.

Requirements

Post curing unit: Sonolux Flash Evo

Basic Material Properties

See technical datasheet (TDS). Available on request.

Specific Manufacturing Considerations

Caution

During the application of Sonopal soft Lack E we recommend wearing personal protective equipment i.e. suitable gloves, safety goggles etc.

Sonopal soft Lack E is suitable for lacquering BTE earmolds and ITE shells. The lacquer can be applied using a dipping method or with a brush.

Processing

Preparation

The surfaces of the BTE earmold and ITE shell should be free from dust and grease residues. Roughening of the plastic surface is recommended for improved wetting behaviour.

Further processing

Dip the BTE earmold to be lacquered into the Sonopal soft Lack E and allow to drip dry. Take care to ensure that no air bubbles form. For ITE shells with integrated electronics, the lacquer should be applied in a very thin coating with a brush.

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